

# STOMA PROTECTIVE POWDER

(NON-STERILE)

keeps skin dry, and reduces irritation from exudate.







# STOMA PROTECTIVE POWDER (NON-STERILE)

### **Stoma Protective Powder:**

It is intended for use on the peristomal skin to keep it dry, which in turn reduces the irritation of the skin by exudate.

Patients with the ostomy such as colostomy, jejunostomy, ileostomy.

## **SPECIFICATIONS**

Bottle	Description
28 g	
	– - Non-sterile
50 g	
	28 g

### Feature:

- Rapidly absorbs moisture on the skin's surface to keep it dry.
- Can be used between the peristomal skin and skin barrier to form a thin protective film on the skin's surface.
- Shows excellent properties of biocompatibility, being odorless and highly hygroscopic.





SY02028

SY02050

# **HOW TO USE**

Patients with the ostomy such as colostomy, jejunostomy, ileostomy.



Clean and dry the peristomal skin.



SHAKE
Shake the bottle before opening.



Aim the nozzle at the intended site and apply powder by shaking the bottle.

APPLY



Wipe off excess powder gently to form a thin protective film on the skin.

**WIPE** 



6 F., No. 29, Sec. 2, Jhongjheng E. Rd., Danshuei Dist., New Taipei City. 251 Taiwan Tel.:(886) 2-2624-2233 Fax.:(886) 2-2624-2266 www.fortunemed.com Mail:info@fortunemed.com



